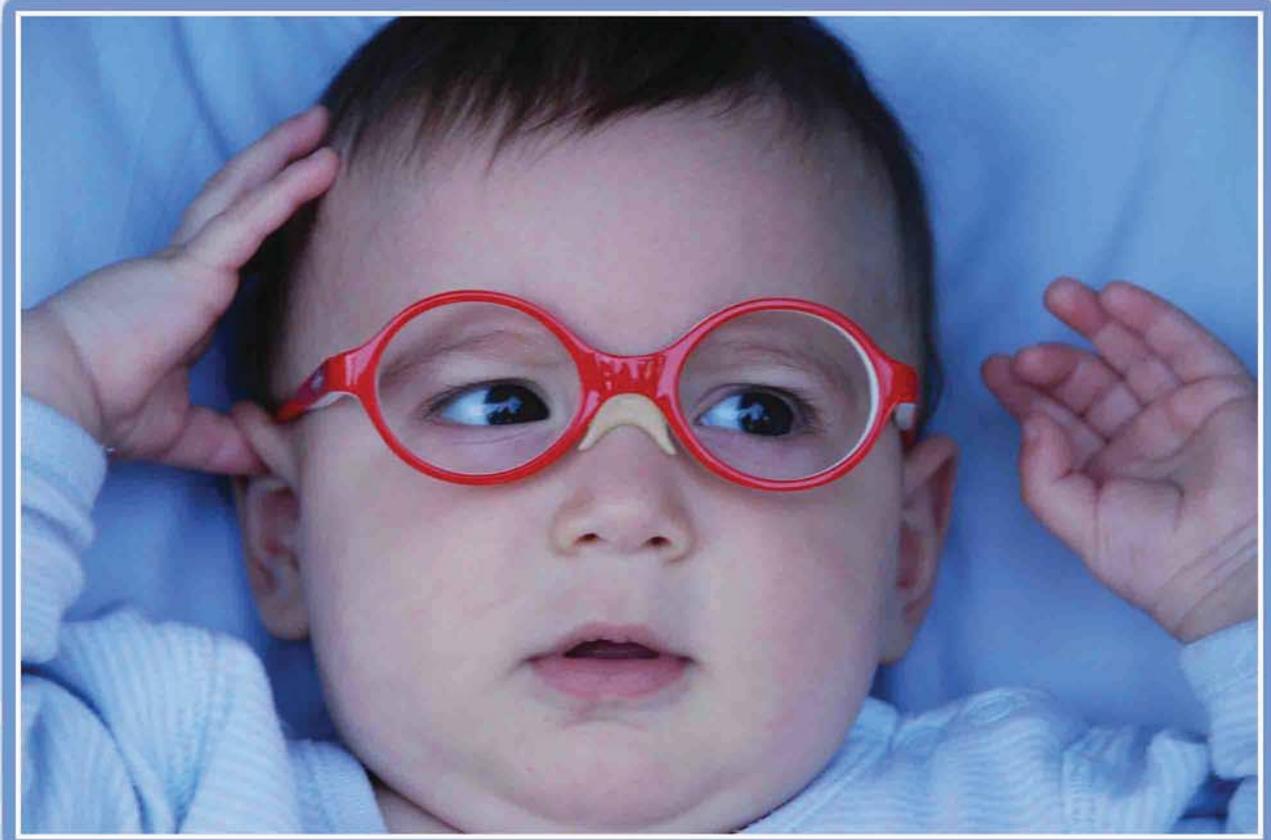
Espace Tropicque

tropique

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Binasal opaque sector glasses on zero power lens.

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By 2 years of age a reversible rame is equipped with an opaque lens and a zero power lens, so that the child does not put his finger in his eye.

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Latent hyperopia is tested with a reversible frame equipped with an opaque lens on one side and a +1.5 on the other side.

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Two far vision tests, calibrated for presentation at 2.50 m distance, appropriate for optimal visual attention. Symbols, pictures and letters are presented by group of 3 in each line to take into account crowding effect occurring with amblyopia. These tests are fit for 2-3 years old infant using the pairing technique. The pairing with the six optotypes is delivered with the test set. The child shows the pattern with his finger.

LETTRES :

TEST CADET VISION DE PRES : *NEAR VISION TEST*

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Specially designed for young children.

It includes : - pictures for children after 2 years,
- numbers for children after 4/5 years,
- reading text designed for 6/7 years of age.

The set includes a pointer to focus the child attention on the line to be read.

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This test "Low Vision" allows the study of low acuity of 1/50 to 4/10 with intermediate acuity between 1 and 2/10ème. It is designed for a presentation of letters to 2.50 m and has a board of appareiment supplied with the test.

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By the 4th month stereo vision is present and can be tested by 7-8 months of age. It is the best test to screen for strabismus and amblyopia.

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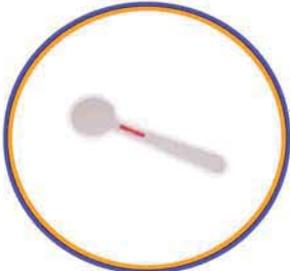
To test ocular pursuit from birth onward with a large bull-eye pattern. To be presented at 30 cm distance around birth and 50 cm after 3 month.

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
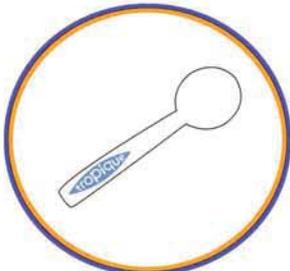
Allows the study of the corneal reflections

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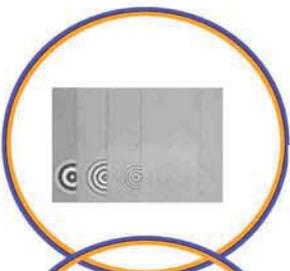screen that allows to observe the eye movement when the screen hidden by the other eye takes the fixing

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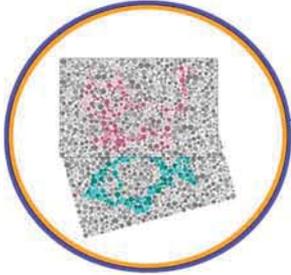
Bébé Vision Acuity Cards are designed to evaluate visual defects from birth onward whatever the origin of the deficit. It can spot very low level of vision. It relies upon the preferential looking technique. The set includes 13 cards and a portable screen. The cards are printed with circular concentric pattern on side, to get rid of astigmatism, and a pinhole in the middle allows observation.

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This is a screening test for color deficiency designed for children 3.5 years of age and older.

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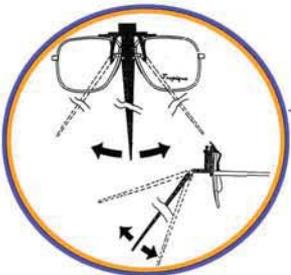
Spherical trial lens, diameter 38, to be inserted in both size of trial frames.

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Box designed by SAVIE. It is for MG, Paediatricians, pediatric nurses & School Nurses. It contains three tests of CADET, two TROPIC screening of strabismus, a screening of TROPIC AV, lamp light, a test relief, and color test

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30cm white plastic rod with gradations in centimeters. It is articulated via a clip permitting provisional fixation for example on the bridge of a pair of spectacles.

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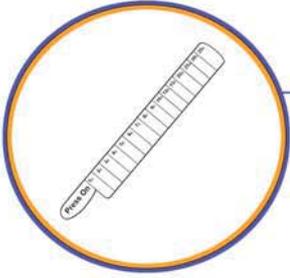
The sundial is a rectangle of black cardboard, with a nasal notch on one side.

The black cardboard isolates the subject from any reference to the ground while the two white lines separate the perceptions of the four hemiretinas.

The sundial is used to diagnose strabismus or hemianopias, whether in the office or at bedside, and in the treatment of strabismus to denervate the affected hemiretina.

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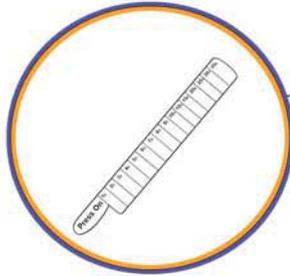
Clear plastic bar containing 12 references Filters (reduced to 1/10th of the VA total occlusion)

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Pack of 2 sheets (4 x 6 cm) that apply on the inside of the glass. 12 references

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Plastic bar containing 16 references to Press On, 1 Δ to 35 Δ

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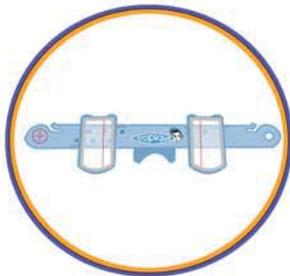
Adhesive bandages. Various sizes. Bag of 20 dressings

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For those sensitive to adhesives. The fabric fits to all glasses

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Easy to measure the pupillary distance or height of small segments of your patients.

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